

DÉESSE^{PRO}



DÉESSE PRO

— OPERATING MANUAL —

CE

DÉESSE PRO

GLOBAL DISTRIBUTOR



Harpar Grace

INTERNATIONAL

Harpar Grace International
Unit 4 Imperial Court, Magellan Close, Walworth
Business Park, Andover, Hampshire,
SP10 5NT, United Kingdom
Tel: 02038 686242
enquiries@harpargrace.com

CONTENTS

The Components	3
Components Explained	4
Operating Instructions	5
Treatment Programmes	6
Specifications	7
Warranty	8
Cautions	8

DÉESSE PRO

THE COMPONENTS




Déesse PRO LED Mask




Protective Eyewear



Control Panel

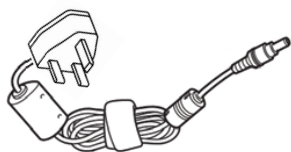


Power Cable

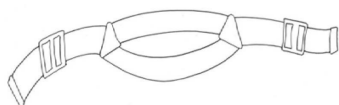

Retaining Strap

*Not required if laying
down during treatment*

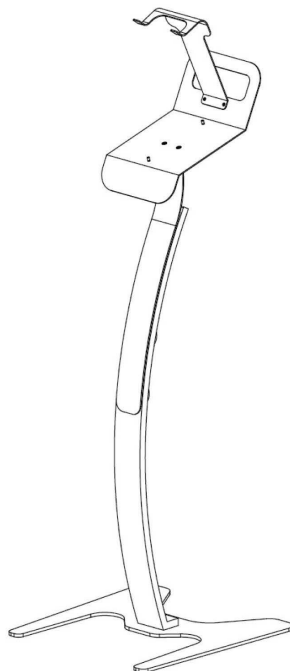

Déesse PRO Mask Stand
(Optional extra)

COMPONENTS


EXPLAINED

Mask Components

Front




Reverse

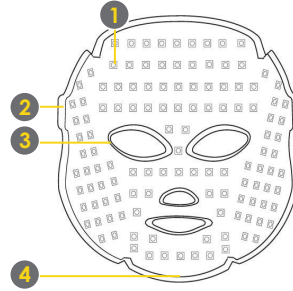


1	Surface mounted LED's	To emit the required wavelength of light
2	Strap connecting holes	To secure the retaining strap to the mask if required
3	Soft rubber eye protector	For safety and comfort during treatment
4	Power Cable	Power Cable to connect the mask to the control panel via the USB port

Control Panel Components

Front

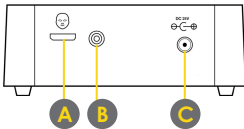



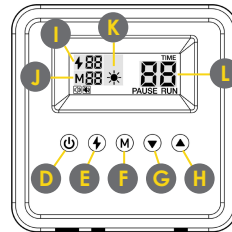



Top



A	Mask connecting USB port	Port to connect the control panel to the mask via the USB cable
B	Power indicator	Bulb illuminates blue when power is connected
C	Power cable port	Port to connect the control panel to the electricity supply
D	Power button	To turn the mask ON and OFF or to PAUSE treatment
E	NIR selector	To engage or disengage Near-Infrared (NIR)
F	Mode selector	To select the required treatment mode
G	Decrease minutes button	Reduce the required treatment duration
H	Increase minutes button	Increase the required treatment duration
I	NIR indicator	Indicates if NIR is selected by Button E
J	Mode indicator	Indicates the mode selected by Button F
K	Action display	Ticks around indicating the programme cycle is running
L	Running time	Indicates time selected or time remaining for the programme

DÉESSE PRO

— OPERATING INSTRUCTIONS —

1. Connect the appropriate end of the 'power cable' into power cable **Port C** at the front of the control panel, and plug the opposite end into the electric wall socket. **Light B** will illuminate indicating that power is being supplied to the control panel.
2. Connect the appropriate end of the 'mask to control panel cable' into **Port A** at the front of the control panel.
3. Press and hold **Button D**  for 2 seconds, and release when you hear a short melody. The LCD screen on the top of the control panel will illuminate. The control panel is now ready for programming.
4. **MODE SELECTION** (see pages 6-7) : Repeatedly press and release **Button F**  until the desired mode is selected.
5. **NEAR-INFARED (NIR)** : Press **Button E**  to set NIR off or on for your treatment.
6. The treatment duration is pre-set, but this can be changed using **Button G**  to decrease treatment time (*minimum 5 mins*), or **Button H**  to increase it (*maximum 30 mins*).
7. Eye protection should be worn over the eyes and under the mask for safety. Place the mask gently over the face. Make sure it is comfortable. The mask can be repositioned during treatment in the event of any discomfort.
8. If laying down, the retaining strap does not need to be worn, but if you wish to sit, then the strap will hold the mask in place. **NB. Please take care not to fasten the straps too tightly!**
9. Press and release **Button D**  to start treatment. The same button can be pressed and released during treatment to PAUSE it, then pressed once again to continue.
10. When the set time has elapsed, the programme will automatically end with a short melody and the LED lights will turn off.
11. The mask should be gently but thoroughly cleansed and disinfected after every treatment using anti-bacterial cleansing wipes (or similar), then allowed to air dry.

NB. Please handle your device and its' components with the greatest of care

DÉESSE PRO

PROGRAMMES

Anti-Ageing



Mode 1

Red

See comment for use of 630nm

Purifying



Mode 2

Blue

Brightening



Mode 3

Green

Post Injectable Procedure



Mode 4

Near Infrared & Blue & Red

PROGRAMMES

Calming



Mode 5

Blue & Green

Anti-Ageing Express



Mode 6

Red

See comment for use of 630nm

**** NB: Modes 1 & 4 utilise 154 red LEDs (630nm). Mode 6 utilises 308 red LEDs (630nm) as express setting.**

DÉESSE PRO

SPECIFICATIONS

Product	Déesse PRO (LED Face Mask)
Function	To treat numerous skin concerns
LED Wavelengths	Blue (415nm), Green (520nm), Red (630nm) and Near Infrared (830nm)
Quantity of LED's	154 of Blue (415nm), Green (520nm), and NIR (830nm). 308 of Red (630nm) Total 770 bulbs
Input	100 - 240VAC, 50/60 Hz, Max 20W
Output	24VDC, 1A

DÉESSE PRO

—USAGE RECOMMENDATIONS—

	MODE 1 ANTI-AGEING	MODE 2 PURIFYING	MODE 3 PIGMENTATION	MODE 4 POST INJECTABLE
INTENSIVE COURSE	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>Immediately post treatment for superior results. Complimenting skin needling & injectables.</p> <p>Treatment duration: 20 minutes</p>
REGULAR USAGE	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Not to be used after:</p> <ul style="list-style-type: none"> • heat based treatments due to the integration of near infrared. • chemical peels • laser treatments • IPL
	MODE 5 CALMING	MODE 6 ANTI-AGEING EXPRESS	NEAR INFRARED	
INTENSIVE COURSE	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 10 minutes</p>	<p>To add NIR to treatment press</p>  <p>Can be combined with modes 1,2,3,5 & 6</p>	
REGULAR USAGE	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Not to used after heat based treatments</p>	

DÉESSE PRO

WARRANTY

- Product must be registered online within one month of purchase for warranty to be valid
- **Register at www.deessepro.com/warranty**
- Warranty is valid for 12 months following date of purchase.
- Should your product become faulty within the warranty period please contact enquiries@harpargrace.com or the company you originally purchased the product from.
- To view our warranty policy visit www.deessepro.com/warranty-policy

DÉESSE PRO

CAUTIONS



OPERATING CAUTIONS

- Use only with the power cables supplied.
- Do not place any wet objects near the device.
- Do not operate with wet hands.
- Do not try to open or tamper with any part of the device.
- Do not make direct eye contact with LED light when device is in operation. Excessive exposure to the light can cause eye damage.
- To protect your eyes from Infrared exposure, wear Protective Eyewear before LED irradiation and remove once the appliance is turned off.



TREATMENT CAUTIONS

- Do not use the mask until you have read the instructions.
- Do not use whilst pregnant
- Do not use whilst being treated for cancer
- Seek medical advice in case of epilepsy or hypersensitivity to light.
- Do not look directly at the bulbs when in use.
- May be used daily if required.
- If in doubt please contact a trained skin practitioner for more information regarding treatment



STORAGE

- Store away from high temperatures and moisture.
- Clean and disinfect with antibacterial wipes after each use.
- Handle with care to avoid accidental damage.

After reading this manual, be sure to keep it in a safe place where the user can always access it.



SENBITEC Co., Ltd.

202,203, 46-24, LS-ro 91beon-gil, Dongan-gu, Anyang-si,
Gyeonggi-do, 14119, Rep. of Korea

EC Declaration of Conformity

We herewith declare that the under-mentioned products are in conformity with the essential requirements and provisions of Council Directive 2014/35/EU. All supporting documentation is retained under the premises of the manufacturer.

Product Name: LED Face Mask

Brand Name & Model Name: DEESSE PRO [SBT-MLLT-PRO-2]

Conformity Assessment Route : Annex III, MODULE A, Council Directive 2014/35/EU

Applied Standards : EN 55014-1:2017, EN 55014-2:2015

EN 61000-3-2:2014, EN 61000-3-3:2013

IEC 62471-1:2006

IEC 60335-2-27:2009+A1:2012 in conjunction with 60335-1:2010

incl. Corr. 1:2010 and Corr. 2:2011 + A1:2013

Expiry Date : 2020.04.28



Signature : _____

Place of Issue : Gyeonggi-Do, Korea

Date of Issue : 2019.04.29.

Kim SangMyung / CEO

DÉESSE^{PRO}

WWW.DEESSEPRO.COM

hgi

Harpar Grace
INTERNATIONAL

EXCLUSIVE GLOBAL DISTRIBUTOR

T: (+44) 2038 686242 | E: ENQUIRIES@HARPARGRACE.COM